**NOTE:**

- the instrument kit has an intuitive layout to guide the surgeon through the drill sequence.
- most instruments are available in short / medium / long.
- all instruments and tooling used during the procedure must be maintained in good condition, cleaned and sterilised prior to use. Please consult the Instructions for Use: Southern Implants instrument tray and reusable instruments (CAT-8003 and CAT-8070) for guidance concerning the maintenance of instruments and surgical trays. Please consult the corresponding drill Instructions for Use regarding care and maintenance of drills.


Retractor

I-ZYG-RET-1



Depth Gauges

I-ZYG-DG-1

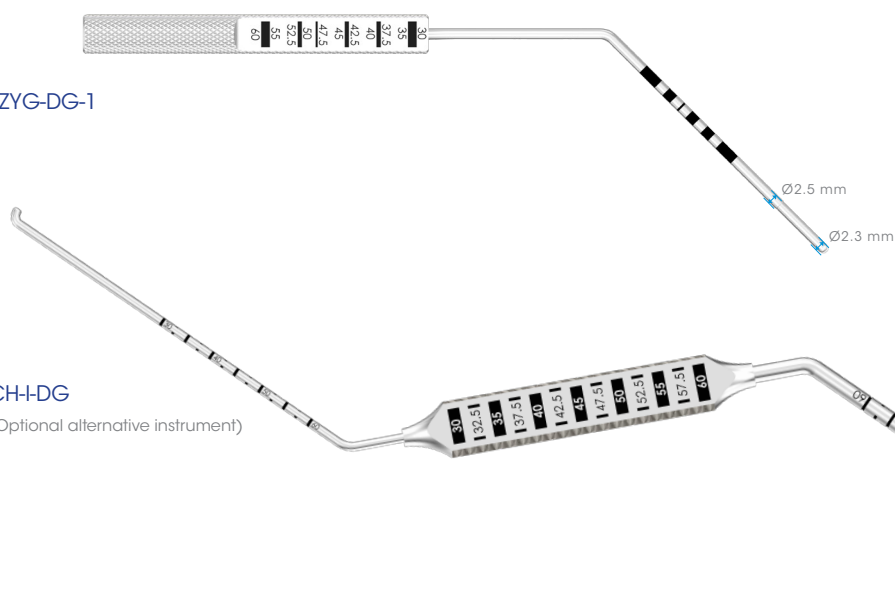

CH-I-DG

(Optional alternative instrument)

Insertion Tools

I-ZYG-INS-1


To fit square on
fixture mount

*only compatible
with the Zygomatic
family

or

I-ZYG-INS-2

To fit square on
fixture mount

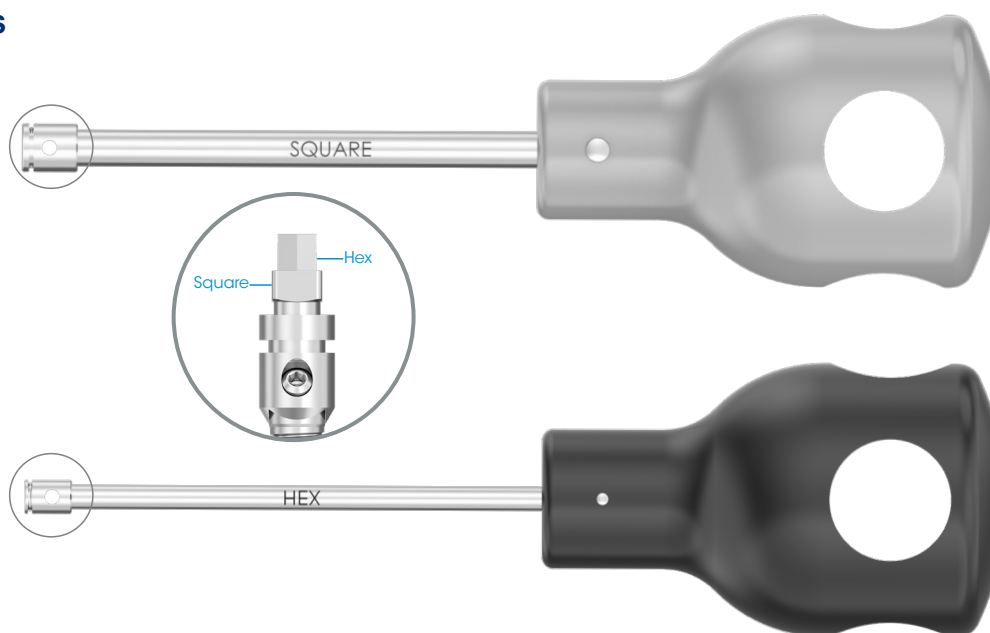

NOTE:

- I-ZYG-INS-1 drives on the square of the fixture mount (silver handle for easy identification).
- I-ZYG-INS-2 drives on the hex of the fixture mount. The narrow hex tip allows for more visibility of the implant head (black handle for easy identification).